# Angelcare®

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AC1100

## WARNINGS

Failure to follow these warnings and the assembly instructions could result in serious injury or death.

This product is designed to aid in the monitoring of your child. When using the monitor, you must still provide the supervision necessary for the continued safety of your child. Always be sure that both the transmitter and receiver are working properly and are within range of each other.

**This product is NOT a substitute for direct supervision of your baby.** Check your baby's activities at regular intervals. Monitoring of premature babies, or those considered to be at risk, should always be carried out under the supervision of a health professional.

**STRANGULATION HAZARD.** Keep the adapter cords out of the reach of children. Do not use with extension cords.

DO NOT place the Nursery Unit or its cord within your child's reach. Do not put the Nursery Unit inside a crib or playard.

Allow for proper ventilation when units are in use. Do not cover the Nursery Unit or Parent Units with any object such as a blanket. Do not place it in a drawer or in any location which would muffle the sound or interfere with the normal flow of air.

DO NOT immerse any part of the Nursery Unit, Parent Unit, Sensor Pad, adapters or docking station in water or any other liquid. Clean only with a dry cloth.

DO NOT place unit near water and moisture. Do not use the Nursery Unit outdoors. Do not use the Nursery Unit near possible wet areas, such as a bathtub, shower, wash basin, sink, laundry tub, swimming pool, wet basement, etc.

Keep Nursery Unit away from heat sources, such as fireplaces, radiators, stoves, and TVs or other appliances. Heat can damage the case or electrical parts.

DO NOT put the parents unit near wireless devices or a microwave oven. Such devices can disrupt the transmission of the alarm signal.

### **WARNING: This product is not a toy.**

Do not use the Nursery Unit if:

- The adapter cords or plugs have been damaged.
- The unit has been exposed to liquids.
- Unit has been dropped or is damaged.

DO NOT OPEN the Nursery Unit, Parent Unit, Parent Unit base, Sensor Pad or adapters. No user-serviceable parts inside. Risk of electrical shock, fire or death.

The antenna used for this transmitter must be installed to provide a separation distance of at least 20 cm (7.9") from all persons and must not be collocated or operating in conjunction with any other antenna or transmitter.

**IMPORTANT:** You must use only rechargeable batteries in the Parents Unit. DO NOT use alkaline batteries or you will severely damage the unit. Charge the batteries for 16 hours prior to first

use. Be sure to have the monitor in the OFF position during this period. Only use the AC adapters included with your monitor. DO NOT use these adapters with any other equipment.

When unplugging the AC adapter from the Nursery Unit, be sure it is switched to the off position. If it isn't, the unit will revert to battery mode if batteries are installed.

You can use rechargeable batteries in the Nursery Unit, however, they will not be recharged by plugging in the AC adapter. To recharge, remove the batteries and use an external charger.

Always place both the Nursery Unit and Parents Unit on a flat surface in an upright position and allow for proper ventilation.

Avoid damage to the Sensor Pad cord and to the adapter cords. Route cords so that they are not likely to be walked on or pinched by items placed on or against them.

DO NOT use this monitor if the Sensor Pad cord and/or the adapters cords are damaged.

Route Sensor Pad cord only below crib mattress and out of reach of children.

Use ONLY AAA (LR03) rechargeable batteries and/or the AC adapters provided for the Parents Unit.

During long periods of non-use remove all batteries from their compartments and unplug the AC adapters.

This monitor uses public airwaves to transmit signals. The Parents Unit may pick up signals or interference from other nursery monitors in your area. Your monitor may even be picked up by other households. To protect your privacy, make sure both units are turned off when not in use.

When the monitor is in use, do not use a crib mobile or other crib accessories that may produce vibrations. Other sources of vibrations may include fans, washing machines, loud music, etc. These sources of vibration may interfere with the performance of your Angelcare® monitor and the alarm will not sound if it continues to detect any movement other than your baby's. Be sure all vibration sources are eliminated before using the monitor.

You may experience false alarms for several reasons. The most common reason is the Nursery Unit has not been turned off after you have removed your baby from the crib. Another reason is if the Sensor Pad has not been placed under the mattress properly. Be sure the Sensor Pad is on a flat, firm surface with the printed side facing up. The sensitivity level may also need to be adjusted. Refer to the Movement Sensitivity Adjustment section.

Always be sure to immediately check on baby whenever an alarm sounds.

Your monitor can be used in any crib that is stationary and designed to safely hold a baby. If your crib has a spring or slat base you will need to place a piece of Masonite board under the Sensor Pad. This support must be at least 13 inches long on all four sides and 1/4 inch thick. Only use your Angelcare® monitor in

a stationary sleeping environment such as a crib where the Sensor Pad can rest on a completely flat, firm surface.

## FCC WARNINGS

Modifications not authorized by the manufacturer may void users authority to operate this device.

**NOTE:** This equipment has been tested and found to comply with the limits for a Class B digital device, pursuant to Part 15 of the FCC Rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation. This equipment generates, uses and can radiate radio frequency energy and, if not installed and used in accordance with the instructions, may cause harmful interference to radio communications.

However, there is no guarantee that interference will not occur in a particular installation. If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one or more of the following measures:

- Reorient or relocate the receiving antenna.
- Increase the separation between the equipment and receiver.
- Connect the equipment into an outlet on a circuit different from that to which the receiver is connected.
- Consult the dealer or an experienced radio/TV technician for help.

This device complies with part 15 of the FCC Rules. Operation is subject to the following two conditions:


- (1) This device may not cause harmful interference, and
- (2) This device must accept any interference received, including interference that may cause undesired operation.

**STRANGULATION HAZARD.** Keep the adapter cord and the sensor pad cord out of the reach of children. Because of the serious strangulation risk to infants and toddlers, parents and caregivers **should never place the nursery unit within 3 feet (1 metre) of a crib.**


## Table of content

Installation in the Baby's room.....	3
Nursery Unit set up .....	4
Parents Unit set up.....	4
Testing our Angelcare® monitor.....	4
Parents Unit settings .....	5
Troubleshooting.....	8
Warranty .....	9
Maintenance of your Angelcare® monitor .....	9
Factory settings and technical specifications .....	9


### INCLUDED

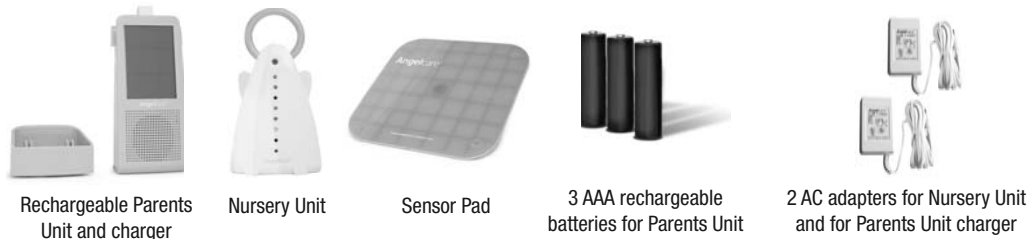
### NOT INCLUDED
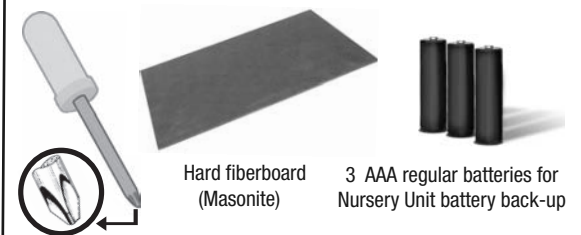
## STEP 1 - Installation in the Baby's room

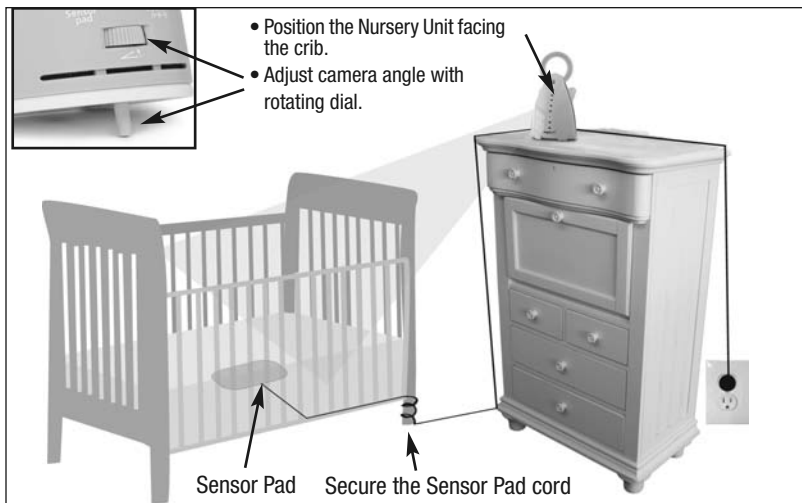
### INSTALLATION OF THE SENSOR PAD UNDER THE MATTRESS

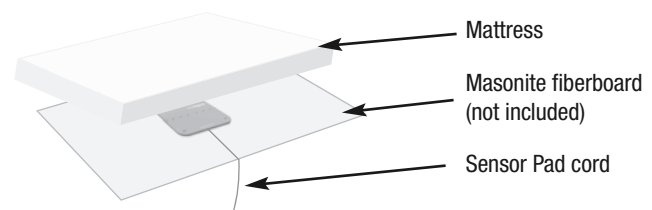
### POSITION OF SENSOR PAD UNDER MATTRESS

1 sensor pad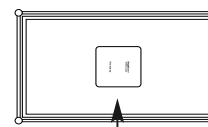


2 sensor pads

(Available in certain countries only)

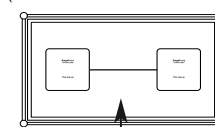- The Sensor Pad must rest on a completely flat and rigid surface. **Place a 6mm (1/4") thick Masonite board to cover the full size of the crib or with minimum dimensions of 28cm (11") X 28cm (11") X 6mm (1/4") between the crib base and mattress, when using one Sensor Pad.** The board will push the Sensor Pad into the mattress and keep the contact between the Sensor Pad and the mattress, should the baby move around. Make sure the Sensor Pad's cord passes underneath the mattress as shown.
- Position the Sensor Pad as shown, **in the center of the crib.** Make sure the printed Angelcare logo faces up. Do not place any bedding between the mattress and the Sensor Pad.
- Secure the Sensor Pad cord to the crib using a tie wrap. **Run the cord on the floor to ensure it is out of the baby's reach.**
- The Sensor Pad works with all baby bed sizes and all mattresses, except memory foam mattresses or mattresses with a hollow frame. You will need to test your Sensor Pad after the set-up is completed.
- **The Nursery Unit can also be installed on a tripod (not included).**


## STEP 2 - Nursery Unit set-up
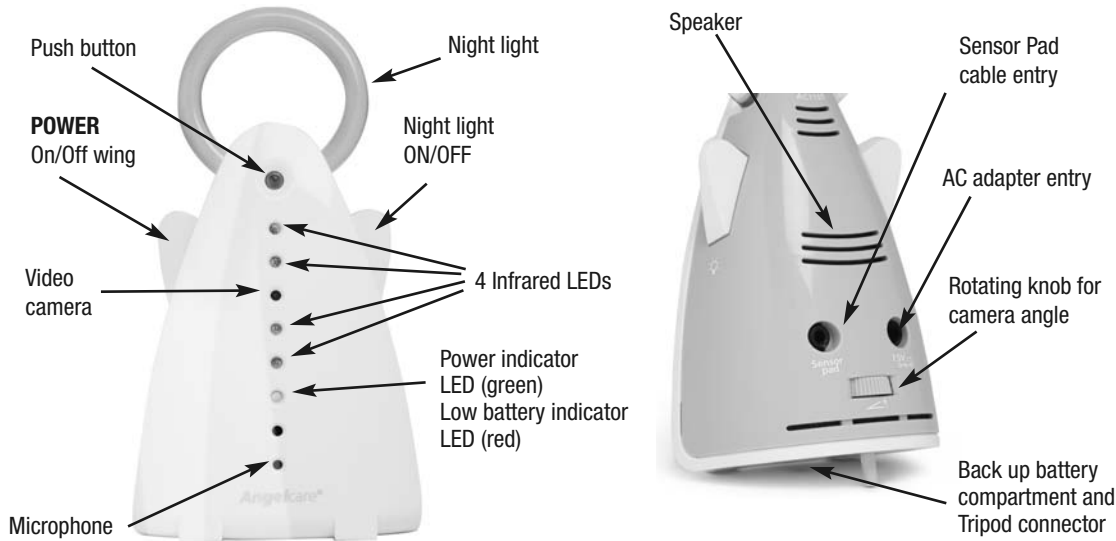
### ACTIONS

- Connect the Sensor Pad cord to the Nursery Unit.
- Insert 3 AAA regular batteries (not included) in the Nursery Unit (for battery backup in case of power failure).
- Connect AC adapter to Nursery Unit and plug adapter to an electrical outlet in the baby's room.
- Push and hold the left wing for 2 seconds to put the Nursery Unit power **ON**.

## STEP 3 - Parents Unit set up

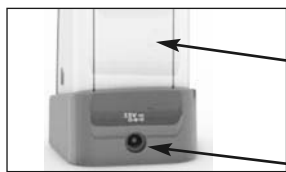
- Remove the belt clip and insert 3 AAA NIMH rechargeable batteries (included) into Parents Unit. NEVER use regular alkaline batteries, they may explode.

- Connect AC adapter to Parents Unit charger and plug adapter into an electrical outlet. **Let charge for 15 continuous hours before using the monitor for the first time.**

- Speaker sound UP
- Speaker sound DOWN
- Press and hold for MUTE
- Talk - Press and hold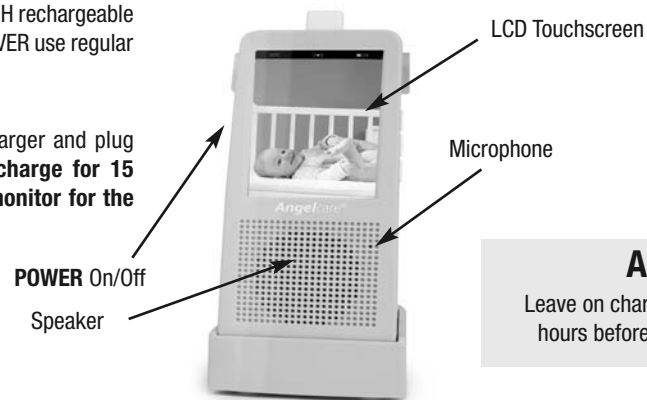

### ACTION

Leave on charger for 15 continuous hours before using your monitor.

## STEP 4 - Testing your Angelcare® monitor

- Test your monitor **BEFORE** using it for the first time
- The Angelcare Monitor can be used with the factory settings or you can modify them in the menu.

**Note:** Test your Angelcare® monitor BEFORE using it for the first time and then on a regular basis to make sure that it is functioning properly. The Angelcare® monitor can be used outside of the crib in a different sleeping area. Test your monitor when moving the Sensor Pad to a new location to ensure sensitivity is adjusted correctly to the new padding or mattress. Refer to the movement sensitivity adjustment section.

**1. TEST THE SOUND** Make sure the sound mode is ON. Have someone else speak into the Nursery Unit or place a radio near the Nursery Unit. You should hear sound from the Parents Unit. If there is no sound, refer to the Troubleshooting section.

**2. TEST YOUR MONITOR TO DETECT MOVEMENT** Make sure the Sensor Pad's cord is plugged into the Nursery Unit then turn both units ON. Make sure the movement function is ON. Gently move your hand on the mattress. The pendulum symbol will move with your every movement. Remove your hand from the mattress. Due to the lack of movement, a pre-alarm "Tic" sound will be emitted after 15 seconds and 5 seconds later, the alarm will sound.

**3. TEST THE STOPPING OF THE ALARM** To stop the alarm, place your hand lightly on the mattress so that the Sensor Pad starts detecting movement again. The pendulum icon on the Parents Unit will start moving again, as movement is detected by the Sensor Pad. You may also press the alarm icon on the Parents Unit, press the push button on top of the Nursery Unit or shut the Nursery Unit OFF to stop the alarm.

**NOTE:** If the alarm does not sound, the Sensor Pad may be detecting a continuous vibration from the floor, a strong draft or the crib being touched. Avoid contact with the crib when the monitor is in use. Place the crib near a solid supporting wall and away from strong air currents. The sensitivity of the Sensor Pad may need to be reduced, consult the movement sensitivity adjustment.

**4. VERIFYING THE BATTERY MODE** Verify that the battery mode is functional by operating both units without the AC adapters. If the green light on the Nursery Unit does not light up, the batteries need to be replaced. If the Parents Unit doesn't light up and the screen does not appear, it needs to be recharged.


## STEP 5 - Parents Unit settings

**IMPORTANT NOTE :** All settings and adjustments done in the main menu and sub-menu screens will not be updated if they are not confirmed by pressing the OK (Confirm/Exit) key.

In the main menu or sub-menu screens, if there is no action done to icon keys for 1 minute, the screen will automatically return to video display. In such case, all settings just adjusted in main and sub-menu will not be saved.

**Note:** When you are in the main menu or settings menu, the sound transmission from the Nursery Unit to the Parent unit is temporarily deactivated. Movement detection will remain on if this option was enabled.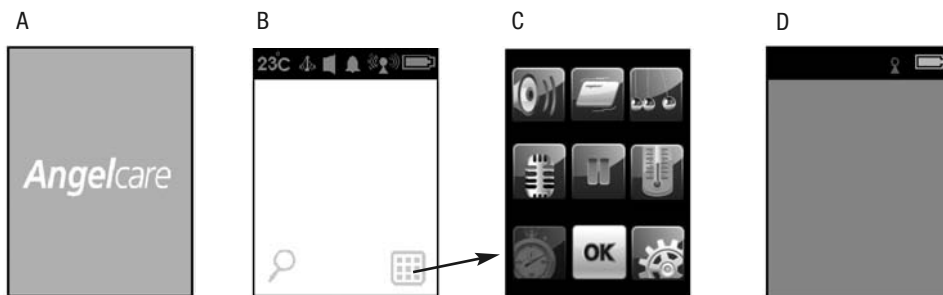

- Press and hold the **POWER** key on the Parents Unit until a blue window with the Angelcare logo appears (A).
- The Parents Unit will then show the screen B with the video capture of the baby's room.
- Touch the small icon at the right bottom to access the menu screen (C).
- If the Nursery Unit is OFF, screen D will appear and the Parents Unit will automatically shut down after 1.5 minutes. In this case, turn ON the Nursery Unit and then turn ON the Parents Unit again.

## 5.1 - Sound


**ON** Sound transmission from the Nursery Unit can be ON or OFF.

Adjust the sound level with the UP (+) and DOWN (-) buttons.

**OFF** Press and hold the DOWN (-) button for at least 2 seconds to MUTE the sound.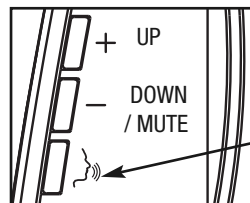

**TALK :** Press and hold to talk to your baby.

Sound icon will appear on the top bar of the video screen when the sound is activated.

Sound ON

MUTE

Level bars will appear on the video screen for a few seconds when you are changing the speaker volume.

## 5.2 - Movement

ON


Movement detection is provided by the under-the-mattress Sensor Pad. This function can be enabled (ON) or disabled (OFF). When enabled, an alarm will be triggered when no movement is detected for 20 seconds. If you are receiving false alarms, you will have to adjust the Sensor Pad sensitivity by pressing the settings icon.

OFF

### SENSITIVITY ADJUSTMENT


When you are using the movement function, you can adjust the Sensor Pad sensitivity in the settings menu, if necessary.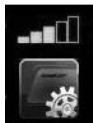

Press the Sensor Pad sensitivity key to the desired level 1 (lowest) to 5 (highest)

Movement icon will appear on the top bar of the video screen when the Movement is activated and the small pendulum will be doing a balance movement.
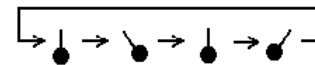
### NO MOVEMENT ALARM

- Triggered when no movement is detected by Sensor Pad for 20 seconds
- Parents Unit and Nursery Unit speakers will emit a loud warning alarm.
- LCD screen display will show the alarm icon on a black background instead of the video display.
- **Check your baby immediately.**
- **To stop the alarm:** press the alarm icon on parent LCD or press and hold the top button of the Nursery Unit and the alarm will stop as shown.


- **To stop the alarm:** press the alarm icon on parent LCD or press and hold the top button of the Nursery Unit and the alarm will stop.

Press alarm Icon

Press and hold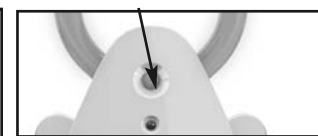

## 5.3 - Tic

ON

OFF

The Tic sound is related to the movement Sensor Pad. It emits a sound on the Parents Unit with every movement of the pendulum telling you that the movement function is on.

This function is only available if Movement monitoring is activated.

## 5.4 - Hold


Press center "hold" icon key to put your monitor on a pause. When the unit is set into 'hold' mode, movement detection and alarm is disabled. Parents Unit speaker emits 3 consecutive beep every 30 seconds as a reminder that the monitor is on 'HOLD' mode. Press icon key to return to normal operation. LCD then returns to normal video display. You can also put the unit on HOLD by pushing the top button of the Nursery Unit and return to normal operations by pushing it again.

## 5.5 - Temperature

Your monitor will always indicate the baby's room temperature. You can be alerted if the temperature is too cold or too hot. If the temperature goes out of the range you have chosen, an alarm will be triggered. Press this icon in the main menu to reach the temperature settings menu.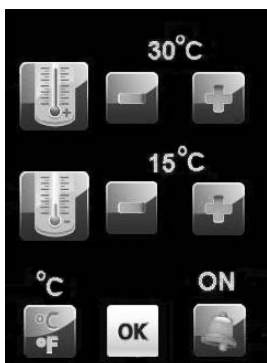

- Press '+', '-' icons at the top row of display to adjust high temperature alarm. (High temperature alarm set range: 22 to 40° Celsius / 72 to 104° Fahrenheit.)
- Press '+', '-' icons in the middle of display to adjust low temperature alarm. (Low temperature alarm set range: 5 to 19° Celsius/ 41 to 66° Fahrenheit)
- Press key at the left bottom corner of screen to switch the temperature scale to Fahrenheit or Celsius. The scale you selected will be indicated above the key.
- If the temperature alarm is set ON, the word "ON" will appear above the alarm icon and the thermometer will appear on top of the video screen bar.

When the temperature detected by the Nursery Unit reaches the limit (high or low) that you have set, the temperature alarm will be triggered: 4 consecutive beeps will be heard every 5 seconds and the temperature alarm icon will be displayed in the center of the LCD screen.

## 5.6 - Vox


ON

The Vox function allows you to put your sound reception on “voice activated mode”. With that function activated, you will only hear louder sounds from the baby’s room. If the function is OFF you will hear the sound in continuous mode. The microphone sensitivity can be adjusted as follows: A low Vox microphone sensitivity means that the sound from the baby’s room has to be louder to trigger sound transmission to the Parents Unit.

OFF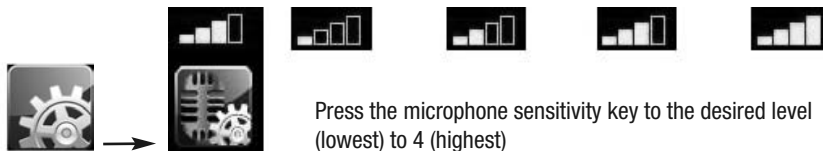

Press the microphone sensitivity key to the desired level 1 (lowest) to 4 (highest)

## 5.7 - Timer

ON  
(Press stop)

When the TIMER is started from the main menu, the screen will go back to video display and show the TIMER running from 0:00 (0 hours and 00 minutes).

OFF  
(Press to start)

The timer has a maximum limit of 5 hours and 59 minutes. After showing “5:59”, it will then stop automatically and disappear from video display.

**Note:** When Parents Unit is turned OFF, the timer will automatically be reset and turned OFF.

## 5.8 - Alarm set up

Press the left icon key in the settings menu to select alarm mode. You can select one of the three following alarm modes:


-  Sound alarm
-  Sound alarm and vibration
-  Vibration only

The selected mode icon will appear in top bar of the LCD video screen

**NOTE:** When the “vibration only” mode is selected, the alarm will not sound on the Parents Unit but it will still sound on the Nursery Unit. When the Parents Unit is placed in the charging unit, vibration is disabled and will temporarily change to sound alarm only mode. During main menu and settings operations, no sound will be transmitted to the Parents Unit but if the Sensor Pad detects no movement, the alarm will sound. The other secondary alarms and notifications are temporarily turned off.

## 5.9 - Video off

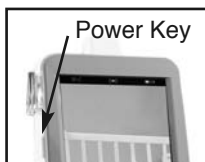Power Key


In order to limit battery consumption, the video display can be turned OFF by pushing the power key. Go to the settings menu to choose if the video screen will shut off after 5, 10, 15 or 20 seconds. If you do not want to use the Video off countdown, press “-” until you see “OFF” instead of numbers over the “-”.

**ATTENTION:** If you push and hold for more than one second, the Parents Unit will shut down.

Will display 5-10-15 or 20 or OFF.

## 5.10 - Zoom mode

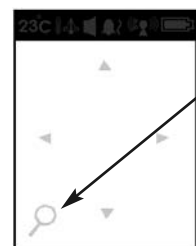
Press the ZOOM icon key in video display to enter ZOOM mode.

In ZOOM mode, the video display is enlarged twice the normal size and the menu icon disappears. Four directions arrows are displayed on the LCD screen. Press on the direction you wish to see enlarged.

To return to the normal video display, press the ZOOM icon again.

## Battery capacity display

Batteries capacity icon is always displayed on the LCD screen. If battery capacity is too low or if the batteries need to be recharged, the Parents Unit will emit a warning beep every 15 seconds. The batteries will support all normal operations until 1/3 full, after which they need to be recharged. The charging cycle display keeps running until the battery capacity reaches a level that allows normal operations. It is preferable to keep the unit on the charger for 16 hours to fully recharge the batteries. While the Parents Unit is put on the charger the battery icon will be running as shown.
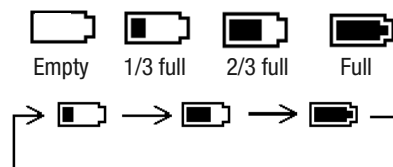
## Link between units

The antenna icon is always displayed on the LCD screen. It indicates the strength of the link to the Nursery Unit. Depending on the quality of the reception it will display the icons as shown. When the link between the Parents Unit and the Nursery Unit is lost, the 'Out of range' alarm is triggered: 3 consecutive beeps will be heard every 5 seconds. The 'OUT OF RANGE' alarm icon will be displayed on the Parents Unit screen. On the top bar, the "lost link" icon will be displayed.

Out of range icon on LCD screen

Good link

Low link

Lost link

**In case the connection between the Nursery Unit and the Parents Unit is completely lost, you can follow these steps to restore the connection.**

1. Make sure the Parents Unit and the Nursery Unit are turned off.
2. Turn the Nursery Unit on.
3. Press and release the left wing 4 times. The night light will flash.
4. Turn the Parents Unit on. A blue screen will appear with the lost link icon on the top bar.
5. While pressing the talk back button on the right side of the Parents Unit, press and release the power key on the Parents Unit.
6. Video will appear on the Parents Unit's screen, meaning that the connection was successfully restored.

## Troubleshooting

Problem	Possible Cause	Solution
<b>False alarms.</b>	<ul style="list-style-type: none"> <li>• Baby was removed from crib and Nursery Unit was not switched OFF.</li> <li>• Sensor Pad cord not connected properly to Nursery Unit.</li> <li>• Sensor Pad is not in full contact with crib mattress, or crib mattress is not resting on a completely flat or rigid surface.</li> <li>• Baby is in a very deep sleep or has moved away from Sensor Pad, to the edge of the crib.</li> </ul>	<ul style="list-style-type: none"> <li>• Turn off the Nursery Unit.</li> <li>• Verify the connection between the Sensor Pad and Nursery Unit. Unplug and re-connect the cord.</li> <li>• Make sure there is no bedding between mattress and Sensor Pad. Sensor Pad <b>must</b> rest on a completely flat and rigid surface. Place a piece of 6 mm (1/4") masonite board to cover the full size of the crib base.</li> <li>• Make sure Sensor Pad is positioned as described under the Installation in the Baby's room section. Sensitivity may need to be adjusted, refer to the Movement section.</li> </ul>
<b>Alarm will not sound.</b>	<ul style="list-style-type: none"> <li>• Nursery Unit is detecting movement when crib is being touched.</li> <li>• Nursery Unit is detecting movement from outside the crib such as motorized devices, strong drafts, etc...</li> <li>• Sensitivity is set too high.</li> </ul>	<ul style="list-style-type: none"> <li>• Avoid contact with the crib when the monitor is ON.</li> <li>• Place crib near a solid supporting wall to avoid Sensor Pad from detecting movement from outside crib.</li> <li>• You may need to decrease the sensitivity, refer to the section Testing your Angelcare® monitor and the Movement section.</li> </ul>
<b>No sound is being transmitted.</b>	<ul style="list-style-type: none"> <li>• Units are too far apart (The out of range alarm will beep).</li> <li>• Batteries are low or improperly installed.</li> <li>• AC adapters are not connected properly or electrical outlet is out of order.</li> <li>• One of the units is not ON.</li> </ul>	<ul style="list-style-type: none"> <li>• Bring Parents Unit closer to Nursery Unit.</li> <li>• Check/replace batteries or recharge Parents Unit.</li> <li>• Check connections or change to another electrical outlet.</li> <li>• Ensure both units are ON.</li> </ul>
<b>Nursery Unit power ON indicator (green light) does not light up.</b>	<ul style="list-style-type: none"> <li>• AC adapters are not connected properly or electrical outlet is out of order.</li> <li>• Nursery Unit is not ON.</li> </ul>	<ul style="list-style-type: none"> <li>• Check connections or change to another electrical outlet.</li> <li>• Push and hold the left wing for 2 seconds to turn the Nursery Unit ON.</li> </ul>
<b>Very weak reception</b>	<ul style="list-style-type: none"> <li>• Units are too far apart.</li> <li>• One or both units are placed near a large metal mass.</li> <li>• One or both units are not in the upright position.</li> <li>• Batteries are low.</li> </ul>	<ul style="list-style-type: none"> <li>• Bring units closer together.</li> <li>• Change the position of one or both units.</li> <li>• Place units on a flat surface away from baby's reach.</li> <li>• Replace the batteries or recharge Parents Unit.</li> </ul>
<b>Static, distortion, interference (interference from other monitors, wireless phones, walkie-talkies, etc...)</b>	<ul style="list-style-type: none"> <li>• Parents Unit located near motorized devices, fluorescent lights, television, etc...</li> </ul>	<ul style="list-style-type: none"> <li>• Rotate Parents Unit or move it away from source of interference.</li> </ul>
<b>Feedback (loud noises emitted from Parents Unit).</b>	<ul style="list-style-type: none"> <li>• Units are too close to each other.</li> </ul>	<ul style="list-style-type: none"> <li>• Move units farther apart (at least 3 meters/10 feet apart).</li> <li>• Decrease volume on Parents Unit.</li> </ul>
<b>The Parents Unit screen is black.</b>	<ul style="list-style-type: none"> <li>• The Parents Unit has been turned OFF.</li> <li>• You may have accidentally pushed the power key and entered the energy saving mode.</li> </ul>	<ul style="list-style-type: none"> <li>• Turn the Parents Unit ON by pushing and holding the power key for 2 seconds.</li> <li>• Press the power key again or touch the screen to display the video again.</li> </ul>

## Limited Warranty

Angelcare® warrants to the original owner of this product that Angelcare® rechargeable video, movement and sound monitor (model AC1100) is free from any material and workmanship defects, for a period of one (1) year from the date of the original purchase. If this monitoring system fails to function properly when used as directed under normal conditions within the one-year period, Angelcare® will either repair or replace the product, at our discretion, free of charge. Please contact your distributor.

The product must be accompanied by a proof of purchase, either a bill of sale or other proof that the monitoring system is within the warranty period. Angelcare® will bear the cost of repairing or replacing the product and shipping it back to you.

**This warranty does not apply to a product which has been damaged as a result of improper maintenance, an accident, improper voltage supply or any other form of misuse. The warranty is also void if the owner repairs or modifies the product in any way. Angelcare® is not liable for any incidental or consequential damages with regard to this product. The warranty also excludes any liability other than what is stated above. No other warranty is given.**

LEGAL RIGHTS VARY FROM COUNTRY TO COUNTRY. SOME COUNTRIES DO NOT ALLOW THE EXCLUSION OR LIMITATION OF INCIDENTAL OR CONSEQUENTIAL DAMAGES, SO THE ABOVE LIMITATIONS MAY NOT APPLY TO YOU.

WE ARE NOT IN ANY WAY RESPONSIBLE FOR ANY OUTCOME RESULTING FROM THE USE OF THIS PRODUCT.

**IMPORTANT!** This Angelcare® monitor is a personal care product. Please do not return this product to the store. If you have questions or need help, please call your distributor before returning this product.

## Maintenance of your Angelcare® monitor

- **RETAIN INSTRUCTIONS:** Keep the user's manual for future reference.
- **WATER, MOISTURE AND HEAT DAMAGE:** Keep all components away from water (sink, bath tub, swimming pool) and places that emit heat (stove, radiator).
- **VENTILATION:** ALWAYS place both units on a flat surface in the upright position so that air can flow freely around them. The Sensor Pad is made of plastic and does not permit the air to circulate. Therefore, mildew build up could develop underneath the mattress in the Sensor Pad area. We suggest rotating the mattress regularly to prevent any mildew.
- **AC POWER SOURCE:** Use only the AC adapters supplied with the Angelcare® monitor.
- **CORD PROTECTION:** Avoid damage to the cords of the Sensor Pad and adapters. Route cords so that they are not likely to be walked on or pinched by items placed upon or against them.
- **CLEANING:** Disconnect all units before cleaning. DO NOT immerse in water. Use a dry cloth to keep monitor free from dust. DO NOT spray with cleaning products or solvents. Wipe Sensor Pad with a cloth moistened very slightly with an antiseptic or mild detergent.
- **OBJECT AND LIQUID ENTRY:** Care should be taken so that objects do not fall onto units and that liquids are not spilled into either unit or Sensor Pad through its openings.
- **NON-USE PERIODS:** When not using the monitor for any long period, remove the batteries from both units and disconnect the AC adapters from electrical outlet.

## Factory settings and technical specifications

Sound Mode .....	ON
Movement Mode .....	ON
Tic .....	ON
VOX .....	ON
Parent unit speaker volume .....	LEVEL 3 OF 6
VOX level (microphone sensitivity) .....	LEVEL 2 OF 4
Sensor Pad sensitivity .....	LEVEL 3 OF 5
LCD .....	ON
Temperature scale .....	Celsius
Timer .....	OFF
Temperature Alarm .....	OFF
High Temperature Alarm .....	30 DEG. C
Low Temperature Alarm .....	19 DEG. C
Alarm mode .....	Sound alarm only

**Range:** up to 655 ft. (200m)

**Number of Channels:** Europe: 64 channels, USA/Canada: 19 channels

**Power Supply:** Three AAA (LR03) alkaline batteries (not included) or 7.5V AC adapter for Nursery Unit.

Three AAA (LR03) rechargeable batteries and 7.5V AC adapter for Charging Base for the Parents' Unit.

**CAUTION:** To reduce the risk of electric shock, do not expose this product to rain or moisture. Danger of explosion if battery is incorrectly replaced. The included batteries are 3 AAA NIMH rechargeable batteries manufactured by GPI International Limited, GN60AAAHC, 1.2V 600mAh. **They may only be replaced by 3 AAA NIMH rechargeable batteries.**

**Batteries:**


- Do not mix old and new batteries

- Do not mix alkaline, standard (carbon-zinc), or rechargeable (ni-cad, ni-mh, etc) batteries

**Transmission:**

Data transmission between the Parents Unit and Nursery Unit is made on the 2.4GHz frequency.

**ErP COMPLIANCE**

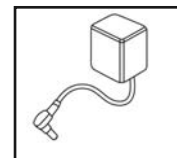**UK – Malta – Cyprus – Singapore**

Use only Angelcare brand power supply  
- model number T07505B001

**EU**

Use only Angelcare brand power supply  
- model number T07505V001,  
T07505F002 (Germany)

**Protect the environment!**

This electronic equipment contains reusable material and must not be disposed with domestic waste. Please take it to a collecting point for electric and electronic equipment (ask your local authorities for details).

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*The ANGELCARE trademark is a registered trade mark of the Angelcare Development Inc.